

please read this manual carefully
and understand thoroughly before
you operate system.

multifunctional LED beauty
machine



User Manual

Connects

Product Series -----	3
Safety Notice -----	4
Operation Notice -----	5-6
Summarize And Theory -----	7-8
Applicable -----	9
Function -----	10
Accessories -----	11
Operation Instruction -----	12-13
Operation Steps -----	13
Operation Notice -----	14
After sale service -----	15

1. Safety Notice

1.1 Please don't disassemble the equipment or try to do some other operation, which haven't been indicated in our instruction. All the repairing work should be done by our admitted professional personnel

1.2 Please don't assemble or operate this equipment while you are near the water or your hands are wet. Notice that don't pour the liquid on it.

1.3 Please pull out the plug from the socket when you met such conditions as below. And in touch with professional maintenance Station.

- A. The equipment touched liquid**
- B. The equipment has abnormal smell, smoking or loud and strong noise.**
- C. Cable is broken**
- D. The equipment is fallen down and broken**

1.4 Please don't put anything on the cables, don't place the equipment at any place where someone can step on the cable.

1.5 Please pull out the power cable after using it for your safety

1.6 Don't put anything into the equipment's jack, otherwise it may cause fire or touch the electricity. If something enters the equipment, please don't pick it out by yourself, contract distributor or our company immediately

1.7 Please don't put the equipment on the anything that can't stand steadily, avoid it turns over and damage.

1.8 Our company has the right to modify the information in this introduction before notify! And keep the explanations right for the above.

2. Operation Notice



Before operation, please read the following content.

2.1 It is essential to make different setting according to specific treatment demand.

2.2 The application of The machine must never be painful and cause redness to the skin or leave marks.

2.3 Move the treatment heads slowly over the skin of the body or the face. Don't letting them stop in any one place on the skin.

2.4 Do not apply the device on children less than 7 years old. The treatment to children over 7 requires medical advice.

2.5 Do not use this device if suffering from a fever, inflammation, dermatitis (acne, eczema, herpes), warts, open or recently stitched wounds, serious and progressive illnesses (cancer, AIDS, hepatitis, blood diseases, etc. or blood coagulation problems. Do not apply the device if you have or have had phlebitis, after veins surgery within the last two months, if you have varicose veins or tingling in your limbs, if you have any concerns consult a physician.

2.6 Never use the device on a pregnant woman.

2.7 The machine could be used on all external and healthy areas of the body except the eyes, throat, ears, glottis and the genital area.

2.8 Must not be applied inside the mouth, or on mucous membrane, or on nipples, or moles without medical advice.

2.9 Never use the device if you suffer from epilepsy. The device must not be used by any person suffering from mental illness.

2.10 In the case of long-term treatment with drugs such as anti-inflammation, anticoagulants, antibiotics or if your skin bruises at the slightest shock, ask advice of your doctor/physician before using.

2.11 Never apply to the body parts with jewelry or ring.

2.12 Regular use of The machine enhances well-being; it is better not to stop its use for a prolonged period of time. Not following operating instructions and usage recommendations may bring lesser results than expected. Do not use the device longer than the recommended time.

2.13 If your symptoms have no improvement, or if in any doubt on the use, consult your doctor.

2.14 Effect may vary depending on your age, life habits (level of physical activity, diet, etc.).

2.15 The machine can be used after any plastic surgery only while following the instruction of the surgeon.

2.16 The machine is designed for professional use only, any personal use of this device is prohibited.

2.17 The Machine continuous working time shall not exceed 2 hours. The best way is cut the power off, let the machine have a rest for 10-15 mins. After the machine have worked for a hour.

3.Summarize And Theory:


Praised as “Magic light”, Photon dynamical instrument is a biological active cold light, and not produce high heat. It applies low-energy and unique optical technology to skin care and treatment, natural and gentle; has prominent effects, low fatalness and no side effect. It is skin nurse specialist, is fit for all kinds of skink, especially useful for seriously irritated skin, acne skin and has notable effect for those in sub healthy state to recover and do skin protection and treatment.

Photon dynamical nurse therapy is a new technology, using the photon power to do skin maintenance and treatment, transfer light energy to cell energy, then accelerate cell growth and blood circulation, stimulate fiber cells to produce collagen proteins, increase skin elasticity and take important part in rehabbing aging and acne skin, lightening flecks, skin upgrade and tighten and easing burnable skin.

Technology:

The application of advanced cosmetic technology, using "point to the surface" ultra penetration techniques, positioning, fixed layer, quantitative and high-energy primary fluid, directly to the skin's dermal layer and the cell interior, so that contains a variety of nutrients and active ingredients The high-energy primary fluid, was quickly organized and cells lining the skin absorption, play a direct role in-house, resulting in efficacy, promote skin metabolism, to maintain the skin whitening and flexibility.

Light Resource Introduction and Usage:

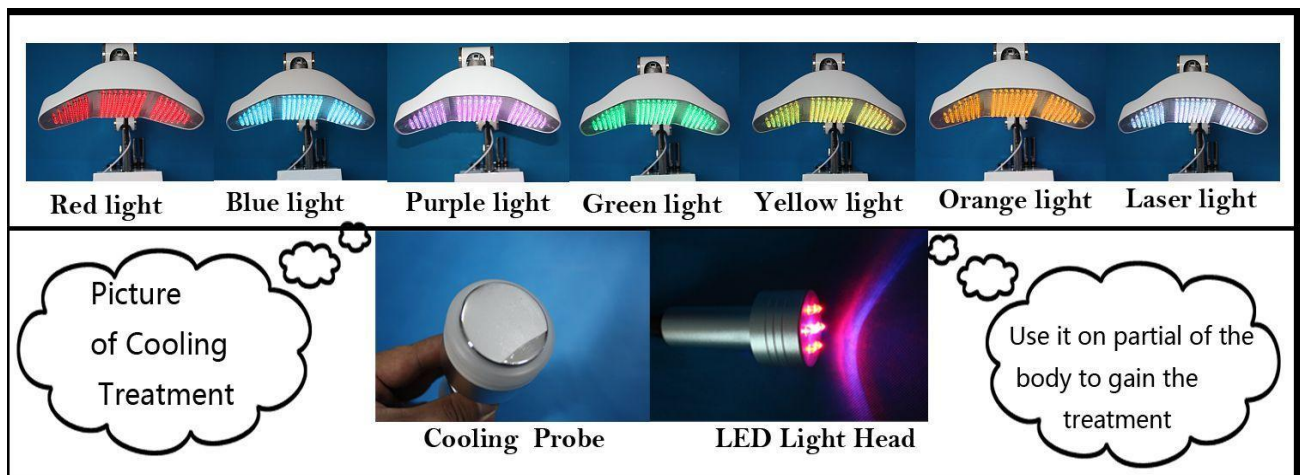

- **Red light**

The red light has the characteristics of high purity, strong light source, energy density well-proportioned, has notable effects in skin care, health care and cure, named as biologic active light. The red light can enhance the cells activity, promote cells metabolism, make skin excrete large number of collagen protein and fiber tissue to fill on their own; accelerate blood circulation, increase skin elasticity and improve withered and yellow, dark condition of skin. Then achieve the efficacy of anti-aging, anti-oxidation and repairing skin, has a effect that traditional skin care cannot comparable. It is especially useful for people in sub healthy state, dry skin and allergic skin. What's more, the nursing process is simple and convenient, comfortable, moderate and effective.

Main effects:

Whitening and lighten spot, tendering skin and dispel crinkles, repair damaged skin, heal slender wrinkles, tighten pore, and proliferate collagen protein.

- **Blue light**

The blue light has the effects of restrain inflammation. Acne is mainly caused by acid bacillus; the blue light can destruct these bacteria with no injury to skin tissue, decrease acne as much as possible, decrease and cure acne during inflammation period in very short time.

Advice for period of treatment: Once every three days, 15 minutes each time.

- **Purple light**

Red and blue light is the dual-band light; it combines both the effectiveness of light therapy, especially useful in treating and repairing acne and acne-scars.

- **Yellow light**

Supply energy for skin cells, promote gland effect, assist digestion, cure skin diseases and enhance immunity ability.

- **Green light**

Has the effects of neutralization, balancing and calming, it can ease strain, dredge lymph and useful in edema elimination.

- **Orange light**

Increase cells energy in proper sequence, it can accelerate metabolism effectively.

- **Laser light**

Penetrate deep layer of skin, accelerate active tissue's metabolism, decompose splash, improve fine lines and flabby skin.

4.Applicable to the crowd:

1. Facial skin treatment

(1) Oil, Coarse pores, Thick cuticle

Purify the skin, remove excess thick horny.Pore purification, decomposition of fat, smooth and tighten pores, restore bright and smooth muscle

(2) Color dull, tired, pale, edema

Oxygen skin, speed up the oxygen absorption, improving circulation.

(3) Relaxation, water scarcity, wrinkles, dry

Upgrade Firming, tightening loose skin, deep hydration and stimulate collagen formation. Updated skin, stimulate cellular intima and accelerate regeneration of cell division and restore young skin condition

2. Eye Beautification

Elimination of dark circle, puffiness, eye appearance of fine lines, eye fatigue.etc.

3、 Body nursing

(1) Cellulite

Repel cellulite, smooth body skin

。 (2) Edema

Edema subsided frivolous, improve blood vessel, lymph circulation. (3) Loose skin after weight-loss

Firming loose skin, enhance the tightening.

5.Beauty Functions:

- 1. Improve symptoms such as aging and loosening skin, big pore, slender wrinkles.**
- 2. Improve pigmentary pathological changes, such as freckle, sunburn, senile plaques.**
- 3. Improve dark complexion caused by bad metabolism or poor circulation.**
- 4. Repair and nurse the damaged skin.**
- 5. Do detumescence, inflammation diminishing and scar elimination treatment for oil acne effectively.**
- 6. For Body Rejuvenation .**
- 7. Elimination of Forehead profile,Corner of the eye pattern,Neck pattern**
- 8. Bio-Moisture Capture.**
- 9. Bio-Leptin skin tone.**
- 10. Bio-fade red skin.**
- 11. Elimination of stretch marks.**

Usage:

- 1. The absorb effect can be better if the nursing is in clean state of skin.**
- 2. Smear corresponding beauty products on skin need cared.**
- 3. Keep the distance between LED light head and skin above 25cm.**
- 4. Usage of LED operating head: Irradiate skin about 10 minutes, then move from cervix out canthus forehead using the lifting method from the bottom up.**
- 5. Usage of laser operating head: The beautician holds laser pen to irradiate skin where there are flecks and pigmentation parts.**

6. Accessories:



◆ RF Head:

Apply the RF technology to make the dispersing molecules get together. Then let the material element go through the skin smoothly. Push the molecular nutrition into skin tissues.

◆ LED Light Handle:

Using red and blue light to do skin phototherapy to customers' chin, ears and other small ranges, so as to increase the elasticity of the skin, improve skin's yellow and dark dumb situation, and improve pigment lesions.

◆ Ultrasonic Head:

To apply the sound waves, enhance the skin penetration, soften the corneous layer, activate the deep skin and cells, promote the blood circulation, speed up metabolism, and make the product better absorbed by skin.

◆ Electric Injection Head for Fluid Product:

According to the customer's care needs with different products, its main role is to open the molecular structure of the skin cells, so that the skin can form the fully open state to import the required energy.

◆ Skin Detection Head:

To use the high definition computer to analyse the customer's skin, then users can accurately analyze the attributes of the skin, such as the pigmentation, and acne etc.


◆ Cooling Probe:

Through the temperature outside alternating with cold and heat, it can make the product of the essence and the curative effect to maximize and let nutrients quickly penetrate into skin.

7.Operation Instruction:

1. Place the fittings well and connect them to main body.
 - (1) Insert output connection wire to the right output jack.
 - (2) Insert LED light head to the right output jack.
 - (3) Insert other probes to the right output jack.
2. Connect the power supply line and turn on power switch.

Insert effective IC card as the LCD screen [1] shows, keep the side which has golden chip up

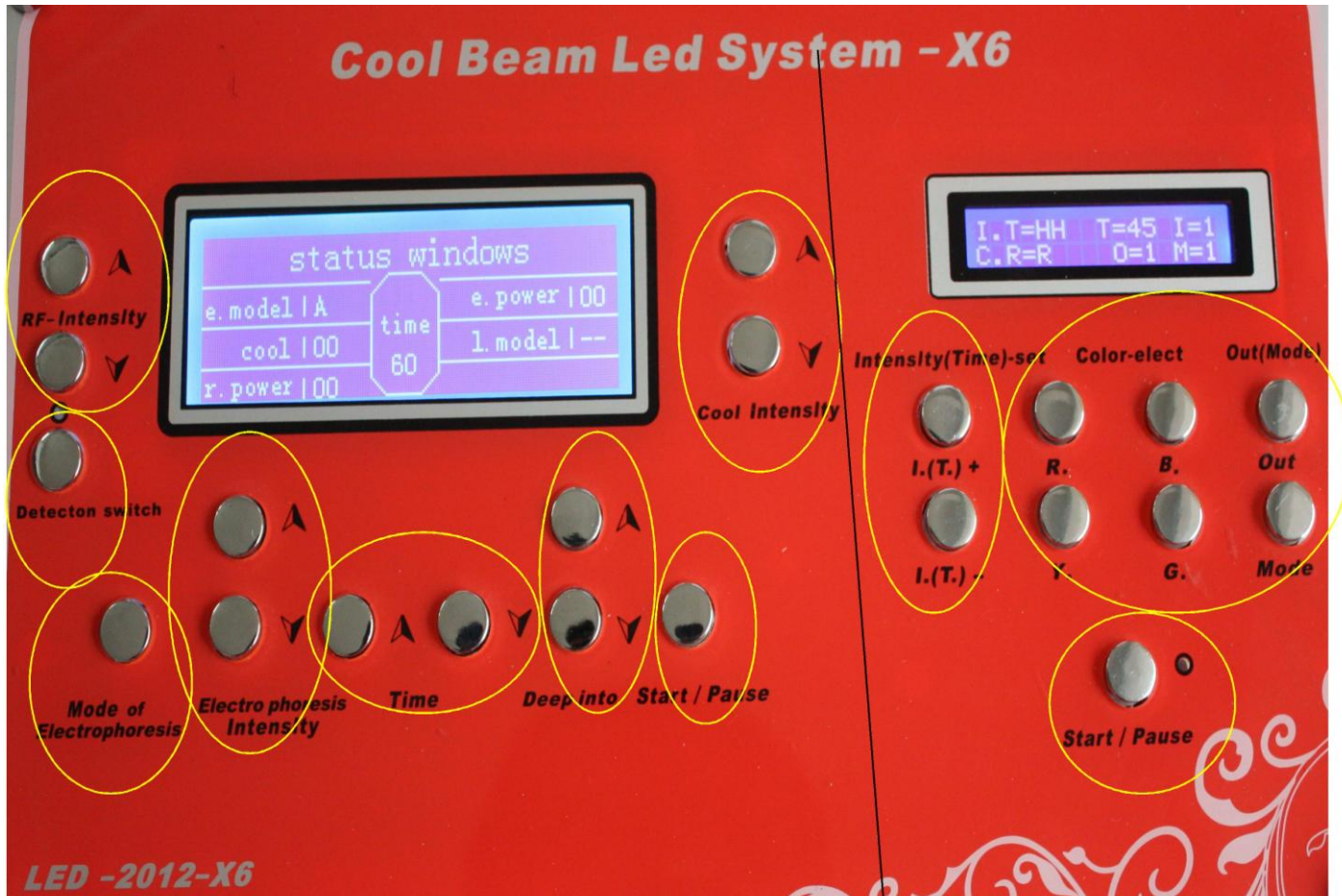

LEFT PART OF THE BOARD (At the LEFT of the black line in the picture):

1. RF-Intensity (Up/Down): Control the strength of RF probe.
2. Detection switch: Control the 50X high definition camera.
(Press it 4 times, then can get a clear picture on the screen)
3. Mode of Electrophoresis: Control the mode of micro current.
4. Electro phoresis Intensity (Up/Down): Control the strength of the micro current.
5. Time: set the treatment time of different probes except of light.
6. Deep into (Up/Down): Control the strength of Ultrasonic probe.
7. Start/Pause: This button control all the probes except light Start and Pause.
8. Cool Intensity (Up/Down): Control the temperature of cooling probe.

RIFHT PART OF THE BOARD (At the RIGHT of the black line in the picture):

The LCD displays as follow:

I.T=100 T=45 I=1
C.R=R O=1 M=1

(1) "I.T=***" means IC card remaining time after use, the IC card setting time is 1 time/100 minutes.

(2) "T=45" means the setting time is 45 minutes, the machine's time setting is 1:00~60:00, press [2] button to adjust when the machine is in pause state.

(3) "I=1" means energy of LED lamp, the machine's setting is 1~10, from weak to strong. Press [2] button to adjust when machine is in state of start.

(4) "C.R=R" means the setting light is red light, the machine's spectrum setting is:

R-red light Y-yellow light B-blue light
G-green light RB-purple light RY-orange light RBG-laser light
Press [3]~[6] can set the light you need.

LED lamp had many kinds of light color.

LED operating head had red light, blue light, purple light

(5) "O=1" means photon energy output channel selection

1-LED lamp 2-LED light head 3-N/A Press [7] can adjust.

(6) "M=1" means work modes, 1 stands for invariable light, 2~9 are impulse light, from fast gradually to slow.

8.Operation Steps:

1. Put through the power on the power switch.

2. Choose the light, the key is compound key, one press to pitch on, one more press to cancel. For example, if you want to change the light from "R" to "B", first press B key, it is RB in the screen, press R key again, then the light is "B" light.

3. When the Start/Pause key is on start, you can press "+" and "-" to increase or reduce the intensity, it is from 1 to 9; when the start/pause key is on pause, you can press "+" and "-" to set the time you want for the treatment.

4. Press "out" key to choose the treatment style. There are three options, "1" is the biggest light, "2" is medium light, "3" is the smallest light.

5. In the process, you can change the shining mode as you wanted. Press "Mode", it can range from 1 to 9, 1 is lasting light. If you want to stop working in midway, only to press "Start/Pause" key. Then turn off the power switch.

9.Operation Notice:

1. Do not look at the light straightly when the treatment is accepted, the guests should wear the protective glasses or covered veil.
2. Don't directly watch the high impulse light from the LED treatment head.
3. Do not operate the laser operating head, LED operating head on eyes.
4. Irradiation nurse on partial should be keep for over 15 minutes.
5. Don't use the machine beside the anesthetic or tinder.
6. Before treatment, the skin can't touch alcohol or acet, or other tinder. If the machine is cleared by alcohol, the machine can be started when the alcohol completely volatilized.
7. The treatment head is precise optics apparatus, avoiding shake or impact.

Warning: Injury may occur when the following instruction are not followed

1. Skin wounds, avoid wounds or does not.
2. The person suffering from epilepsy forbidden to do.
3. Heart disease (such as arrhythmia persons), especially in people with cardiac pacemakers forbidden to do.
4. The person suffering from cancer forbidden to do.
5. The person suffering from infectious forbidden to do.
6. The person suffering from pregnant forbidden to do.
7. Major surgery patients, people with diabetes do ban.
8. A person which body have any metal do ban.

10. AFTER-SALE SERVICE AND MAINTENANCE

The term of maintaining guarantee is main device 1 year and spare part 3 months from the date of purchasing according to the receipt. (Expendable is not included in it.) Others not included in maintaining guarantee or over the time, we also supply to maintain and renew but need charges. Please contact with the dealer if need change of purchasing the parts and easily expendable products.

A. We will guarantee the quality of our equipment, normally there is no quality problem

- 1. If there is a deficiency, please email or call us on the reasons, it would be best to dealt with by email or telephone. Because some of the problems are caused by using the machine in a wrong way.**
- 2. We suggest the client to take photos of the area where the problem occurs, we will pass them to our technician , our technician will give a satisfactory answer within 1-7 working days**
- 3. If the problem comes from the spare part within the warranty, we could consider in changing the spare part to our client, which could be sent with his/her next order. If the next order is uncertain, we could pay freight cost to send the spare parts.**
- 4. If the problem comes from the main device within the warranty, we could advice our client to post the device back to our company for repair by ordinary post, or we could re-send a new one to our client. If the cost of posting the problematic device back to BEST is more than the value of a new device, BEST would take it into consideration, according to the actual status; apply to re-send a completely new device back to our client.**

B. We won't supply maintaining guarantee if troubles as below:

- a) Disassembling the machine by self;**
- b) Fall it without caution;**
- c) Lack of reasonable keeping and protect;**
- d) Not operate according to the user manual.**
- e) Expendable reasons**

If changes of the design and specification of the machine, we will not notice you in addition. If any queries, please feel free to contact with the local marketing company.

All in one, we will try our best to serve you and also need your fully cooperation. Thanks.